


KN95 Personal Protective Mask(Non-medical)	
Model	SP-FM007
Type	Earloop
Size	150mm*140mm
Color	White
Materials	Multi-layer Non-woven Fabric + Melt-blown Filter + Ajustable Nose Clip + Ear Band
Standard	GB2626-2006
Certifications	CE, FDA
Validity	3 Years
Storage Temperature	-10°C ~+40°C
Inner Packing	one pc in one plastic bag, 10pcs to a inner box
Outer Packing	Carton

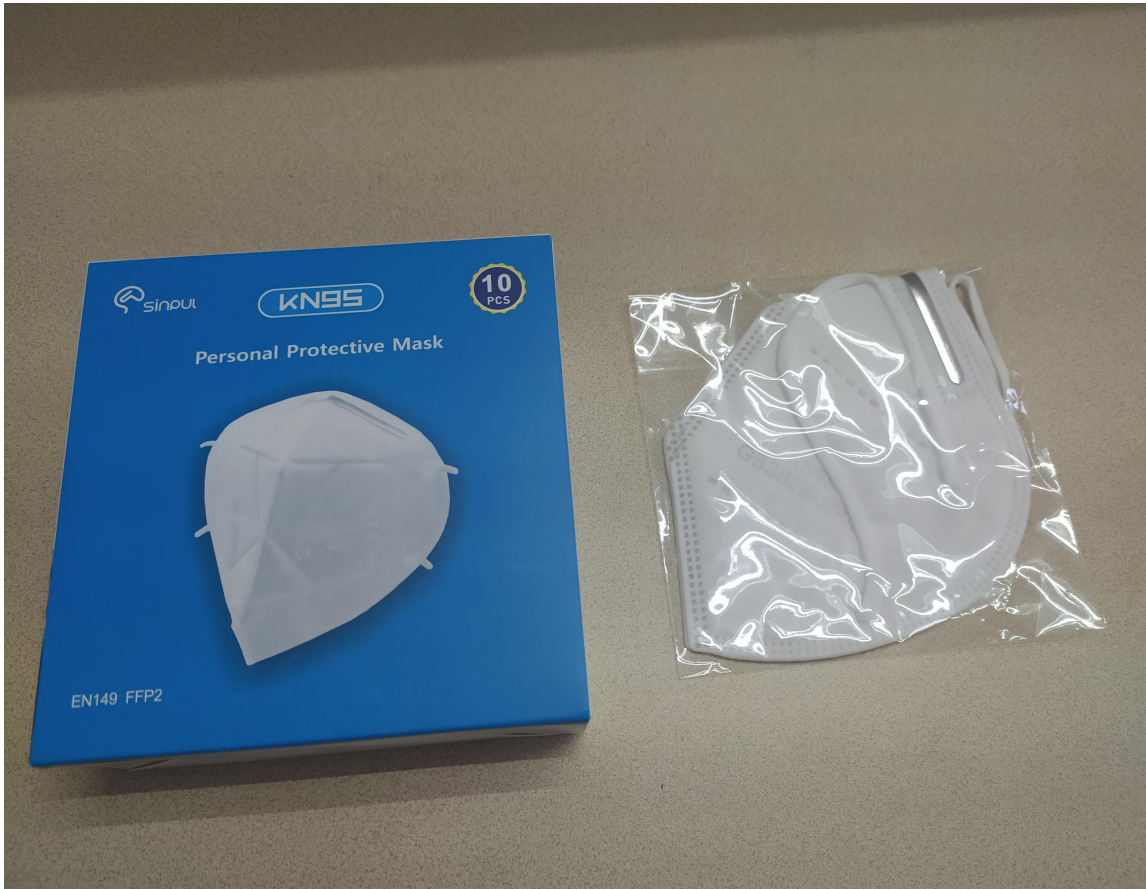



KN95 Personal Protective Mask User Instructions (Non-Medical)

[Name] KN95 Personal Protective Mask

[Model and Specification] Model: SP-FM007 Specification: 150mm×140mm

[Structure] Multi-layer non-woven fabric and melt-blown filter, adjustable nose clip and ear band.

[Application Scope] Personal respiratory protection, filter particles and droplets.

[Performance] This product is in accordance with EN149:2001+A1:2009 ,FFP2 .NR And GB2626-2006.

⚠ Precautions & Tips

1. The mask must be fit on the face tightly without any leakage. If needed, tie a knot to make the mask, fit tightly on the face.
2. Contaminated masks should be disposed as hazardous waste in accordance with national regulations.
3. For personal protective purpose only,not suitable for medical infectious ward or operating room.

[Quantity] See the package

[Product Date] See the inspection label

[Storage Method] Product should be stored in clean, dry condition with temperature range -10°C to +40°C with a maximum humidity of 80%.

[Validity] 3 years

[Name Of Registration] Sinpul Health(ShenZhen)Co.,Ltd.

[Address] A1301,No.54 Jiaxi Road, Central Community, Pingdi Town, Longgang District, Shenzhen City, China

[Contact] 0086-0755-28282339


MADE IN CHINA

Sinpul Health(ShenZhen) Co., Ltd

Address: A1301 NO. 54 Jiaxi Road, Central Community,
Pingdi Town, Longgang District, Shenzhen City, China



HOW TO WEAR

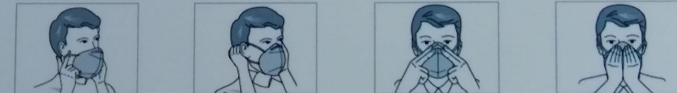

- The nose clip must face upwards(over the nose)while the remainder of the mask covers the face using both hands, pull each ear strap simultaneously
- Place mask against chin and pull the ear straps behind and around the ear until you have a comfortable fit
- Place fingers of both hands in the middle of the nose clip. Press and adjust the nose clip to fit and form the shape of the nose bridge
- Perform a fit check by breathing in and out. While breathing out check for air leakage around your face